




Model: PWD-2300

>Model: PWD-310C

IPL with IPL/IPLSHR/BBR treatment Modes

>How does it work:

Intense pulsed light (IPL) is a type of intensive, broadband, coherent light source which has a wavelength spectrum of 420 nm ~1200 nm. With these special properties, the IPL has a wide application in non-ablative therapies based on theory of human skin tissue's selective absorption and photothermolysis of light sources. Meanwhile, IPL treatment is more effective, with no downtime and can make the patients get recovered more quickly than conventional therapies

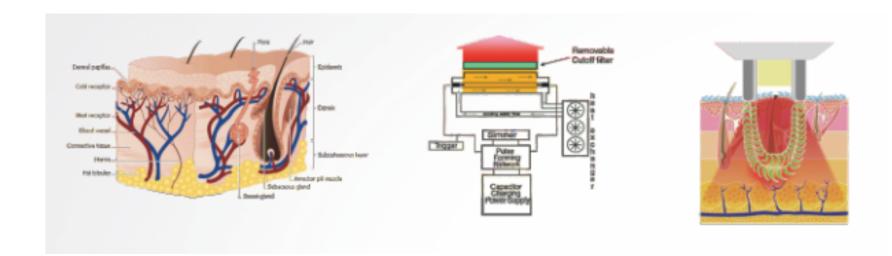

>Comparision normal IPL with SHR:

| Parameter | Normal IPL | SHR | SHR Advantages |
|----------------|-------------|---|--|
| Technology | Traditional | New | It is popular in market now. |
| Frequency | 1 | 1-5Hz | Speed up the treatment, save time |
| Working method | Traditional | Traditional technology & In-motion&BBR | Moving IPL can use less time to do treatment on back, leg and arm. |

>Technical specifications:

| Handpiece | 1*IPL SHR/EPL | |
|---------------------|--|--|
| Spot size | 15*50mm | |
| Wavelength | 420~1200nm | |
| Filter | 420/510/560/610/640~1200nm, SHR filter | |
| IPL energy | 10~60Level | |
| SHR repetition rate | 1~5Hz/ 1~10Hz | |
| RF output power | 200W(optional) | |
| Operate interface | 8" True color touch screen | |
| Cooling system | Advanced air & water cooling system | |
| Power supply | AC 100V or 230V, 50/60HZ | |
| Dimension | 66x46x42cm (Lx Wx H) | |
| Weight | 37Kgs | |

>Support IPL, IPL SHR& BBR mode for same one hanpdiece:

IPL is a type of intensive, broadband, coherent light source which has a wavelength spectrum of 420~1200nm.


The IPL SHR means Super Hair removal. Use newest technology, emit low and equally fluence, treatment is pain-free and save treatment time.

The BBR (Broad Band Light) is actually skin rejuvenation in SHR mode, it use the pulse light energy and gently heat upper layers of skin.






>High quality IPL handpiece:

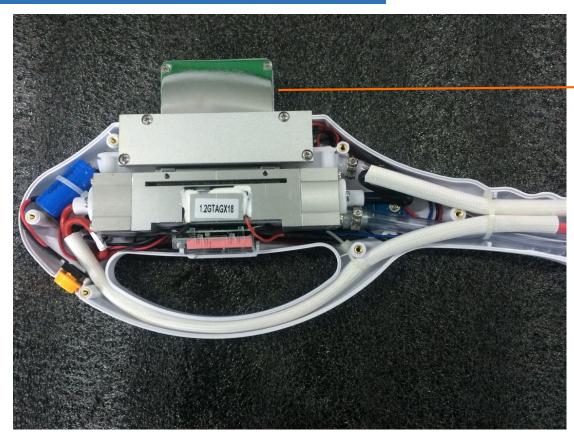

Our IPL Handpiece tretment head is crysal, the material is JJS1 with high hardness, so the cooling result is efficient, other one may use the silica glassses.

Our normal IPI handpiece lamp is from USA origial. Our IPL SHR handpiece lamp from Germany origional, made in Uk. which can ensure long lifetime and stable energe output, most of other CHInese company use only China demestic lamp, which performance is totally different.

>High quality handpiece:

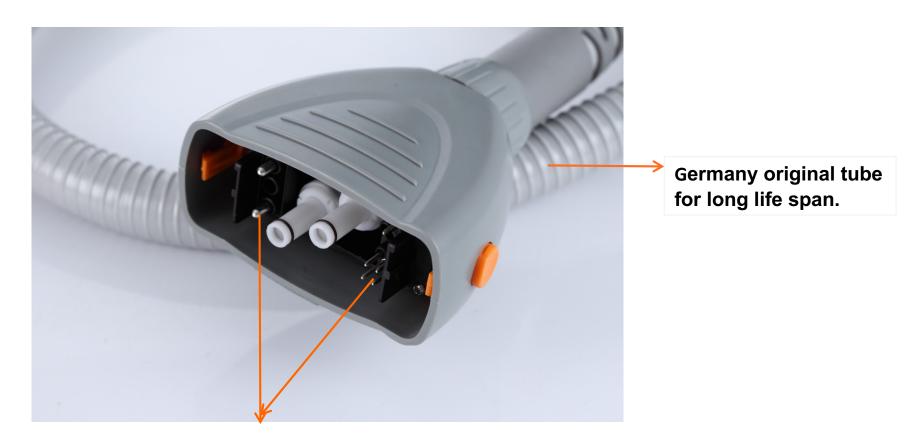

The pins is from Germany original, which can endure high current and high voltage.


>Filters:

We have a complete range of <u>changeable filters</u> instead of the fixed one which is easier of installation and cost-effective; 420/510/560/610/640~1200nm and 610~980nm for SHR.




>Detachable mechanical design:

The casing could be detached to pieces, it's convenient for maintenance.

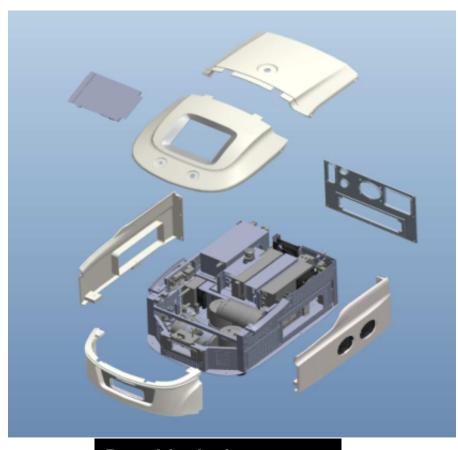

Portable devices

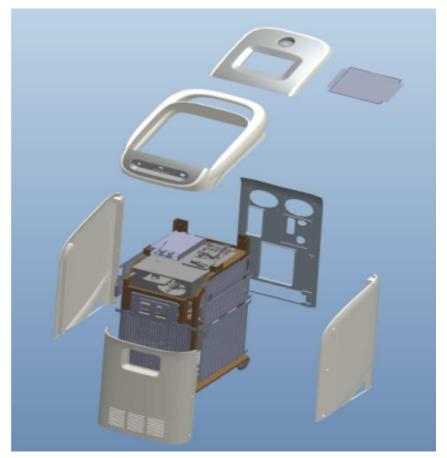

Full Standing Devices

>High performance cooling system:

Make sure to work with long lifetime and high flow water.



DC Pump-3.2L Portable



DC Pump-5L Vertical




Italy water pump-5LOptional

>High performance cooling system

COOPER RADIATOR (stainless steel pipe) +high speed cooling fans are much better cooling effect. It can compete with Almar Laser.




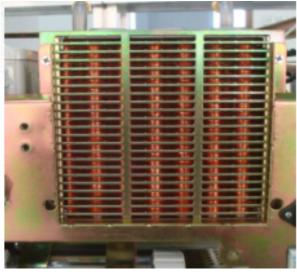



>Water filter design:

Remove impurities and ensure clean water, prolong lifetime of handpiece.

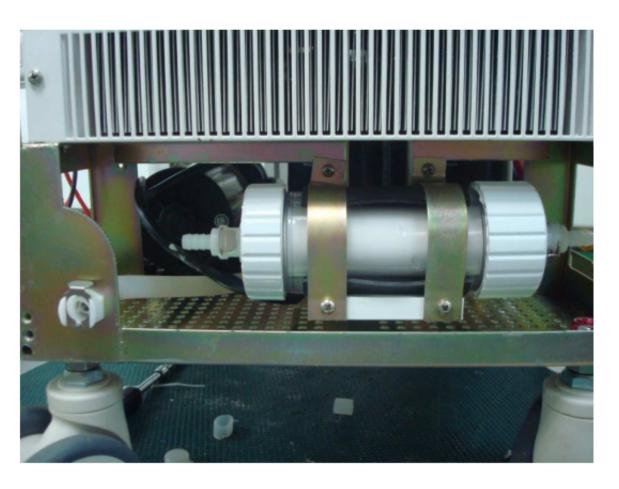

>Advantages: Friendly use operator interface

♦ 8" True color touch screen

Arm-A9(Andoid O/S) with high resolution with standard mode(presettin) and professional mode.





>Advantages: powerful enginer mode/settting mode

◆ Multi-languages supported

Total 16 languages available for the moment. If you want to add more languages, please simply provide us the translation contents.





>Advantages: easy for service

◆Arm-11(Control board)

With advanced **Linux/OS** Operate System, support Software develoment and upgrading/ Database managemen/Powerful factory-setting/Multi-languages

USB function

With USB function you can update the software simply and automatically.

♦ IC card function

To control shot number and expiry period.

With the IC card, you can set the expiry date and shots number to control valid time of system operation. Thus you can have derivative cooperation ways:

- ●1. Rental
- •2. Installment
- •3. Control the after service (Handpiece warranty period)





>About packing

Our IPL handpiece delivery with strong package for safety transportation.



>About packing

◆ Aluminum packing box with wheels and brake, very convenient transportaion.

